**ARTIKEL-NUMMER: 740**  
**Dermatril 740**

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The CE marking on this glove means that it meets the essential requirements for in the European Directive EEC 89/686 concerning Individual Protection Equipment (IPE): Harmlessness-Comfort-Dexterity-Sturdiness. Protective Gloves conform to the EN 420. Approved to EN 374 according to Article 10 of Directive 89/686/EEC. IFA, Alte Heerstraße 111, D-53757 St. Augustin, Identification number: 0121

Surveillance after 11 B of Directive 89/686/EEC, chemical risks according to EN 374: same location as type test. The identification number for testing and certification centre 0121 for EC type testing and monitoring of quality assurance measures relates solely to the content of EN 374-1: 2003 and the PPE Directive 89/686/EEC.

**EN 374, Waterproof gloves and little protection against chemical hazards**

EN 374-3 was rated positively and certified by notified body. When used with chemicals the protective glove should be checked for adequate permeation.

Article	Chemical	Penetration time in min.	Protection index
740 / 741 / 742	Sodium hydroxide 40 %	> 480	6

**EN 374, Protection against bacteriological contamination**

KCL chemical protective gloves have been penetration-tested to the highest performance level (3) set out in EN 374-2. This quality limit equates to an AQL < 0.65.

Article	Name	Size	EN 388
740	Dermatril® 740	6, 7, 8, 9, 10, 11	000X
741	Dermatril® L 741	7, 8, 9, 10, 11	000X
742	Dermatril® LR 742	7, 8, 9, 10, 11	000X

Gloves conform to Regulation (EC) No 1935/2004. Approved for short-term, direct contact with all food types.

Article 742 requirements of EN ISO 14644-1: ISO Class 5, GMP A/B.

**EN 388, Mechanical risks**

1st digit Abrasion resistance	(min. 0; max. 4)
2nd digit Cut resistance	(min. 0; max. 5)
3rd digit Tear strength resistance	(min. 0; max. 4)
4th digit Puncture resistance	(min. 0; max. 4)

X = Cannot be tested

This glove meets the following requirements of EN 455:

EN 455-1: 2001-01;	completely
EN 455-2: 2011-2-05;	completely
EN 455-3: 2007-03,	5.3 proteins extractable
EN 455-4: 2009-10;	5.2 Accelerated determination of shelf-life

The glove is not a medical device within the meaning of Directive 93/42/EEC.

**WARNING!**

Resistance to the chemicals listed was determined under laboratory conditions and may be adversely affected by changes to physical properties such as temperature, abrasion, stretching, etc.

When using highly corrosive chemicals, degradation is the most important factor in the choice of protective gloves.

These protective gloves provide no protection against extreme cold (< -5 °C), heat (> 50 °C), electricity.

Do not use in the vicinity of moving machine parts. Risk of being drawn into the machine.

Unusable when the gloves are cracked, porous and stiff.

**Allergy Notice:** The Gloves may contain traces of dithiocarbamates.

**Use:** The protective gloves you wear must be of the correct size. Note that using undergloves may result in some usage restrictions. Check the protective gloves for damage before you use them. Prevent the

penetration of pollutants over the edge of the glove. Prevent carryover of contaminants located on the glove and the cross-contamination when removing gloves.

**Cleaning:** The specified gloves are not washable. Do not use any chemicals or sharp implements when cleaning (wire brushes, sandpaper, etc.). Dirt brush off.

**Expiry date:** With proper storage, no reduction in performance levels within 36 months.

**Storage / Transport:** Flat, dry, dark, with no additional weight load in its original packaging, at a temperature of 5 °C - 25 °C. Protect from sunlight and ozone.

**Disposal:** If not contaminated with chemicals, gloves can be disposed of in the household waste. If the gloves have been exposed to chemicals, follow the disposal instructions of the manufacturer of the chemicals.

**For further information please contact.**

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**KCL GmbH**